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Place completed form in the outer pouch of the Sample Bag.

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Before you collect the stool sample, please aim to:

- Maintain regular diet within 2 weeks
- Avoid antibiotics within 4 weeks
- Avoid probiotics within 4 weeks

Do not collect the stool sample when it is watery due to diarrhoea and when there is bleeding from haemorrhoids and menstruation.  
Follow this Guide to collect the sample carefully. Return the sample within 24 hours.

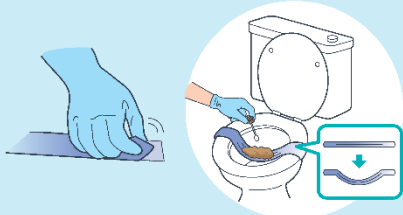
**第一步 STEP 1** 於樣本袋的標籤上填妥樣本收集日期及時間。  
Fill in the sample collection date and time on the label pre-sticked on the Sample Bag.

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When collecting sample, try to avoid contact with urine, toilet water, tissue paper etc. as this may affect test results.


如圖所示，從雙面膠紙撕下膠面，將水溶性糞便收集紙貼在馬桶座板上。  
Remove the liners from the double-sided tapes. Stick the water-soluble Stool Collection Sheet onto the toilet seat as shown.

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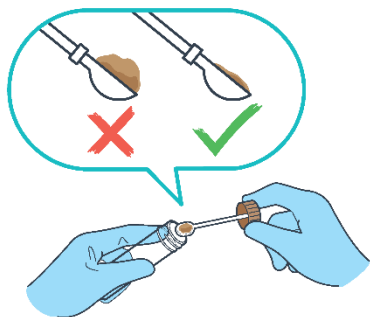
Using the collection spoon attached inside of the lid, add ONE SPOON of stool to the tube. Replace the cap tightly. Too much or too little sample will affect the test results.

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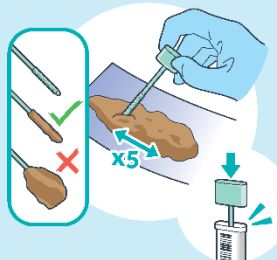
With the sampling probe, scrape the stool surface from side to side for 5 times. Sample will be enough when the grooved part of the sampling probe has been completely filled up. Too much or too little sample will affect the test results. Insert the probe back to the tube and make sure that it clicks closed.

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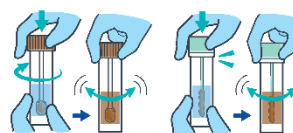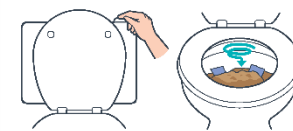
Make sure both tubes are tightly closed. Shake the tubes side to side until the samples are thoroughly mixed with the solutions.

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Flush the water-soluble Stool Collection Sheet (which is flushable) and all remaining stool down the toilet. Dispose the gloves to rubbish bin. Wash your hands with soap.

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Put the sample into the Sample Bag and seal. Ensure the Consent Form is placed in the outer pouch of the Sample Bag. With the checklist on the Sample Bag, make sure all the items are placed in it and mark with a ✓. Put the Sample Bag into the Outer Packing Bag and zip it tightly. Within 24 hours, return the whole set of sample to the designated location.

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- Both Tubes contain preservatives. DO NOT drink or pour away the preservatives. If swallowed, please rinse your mouth, drink 1-2 cups of water and seek medical advice.
- Avoid direct contact of preservatives with skin and eyes. In case of it, rinse with plenty of water and seek medical advice.
- Keep out of the reach of children.
- Store the Kit at room temperature. Keep away from heat and direct sunlight.

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Analysis and report will be sent to your doctor when it is ready. Please consult your doctor for detail on the interpretation of the report.

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M3CRC<sup>TM</sup> ("Test") is not a diagnostic test. The test result of the Test should be interpreted with caution. A test result of "low" to "medium" risk of having colorectal adenoma and "low" risk of having colorectal cancer does not guarantee the absence of colorectal adenoma and/or colorectal cancer. A test result of "medium-high" or higher risk of having colorectal adenoma and "medium" or higher risk of having colorectal cancer does not guarantee the presence of colorectal adenoma and/or colorectal cancer. A test result of "low" risk of recurrence does not guarantee the absence of recurrence. A test result of "high" risk of recurrence does not guarantee the presence of recurrence. The screening interval for the Test has not been established. GenieBiome refers to GenieBiome Limited and its Subsidiaries.

### 聯絡我們 Contact Us

+852 9558 1584  
+852 2903 0700  
crc@g-niib.com

G-NiIB<sup>®</sup> is a registered trademark of GenieBiome Limited.

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Place completed form in the outer pouch of the Sample Bag.  
請將填妥的表格放於樣本袋外袋。

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- 患者之未被用於該檢測的樣本 ("剩餘樣本") 以及相關樣本的殘留物檢測結果可能有助於研究。診斷方法的開發和改進及治療方案。此類措施也有可能未來有助於向患者提供更好的醫學建議及指導。例如診斷和/或治療潛在疾病。Patient's sample which is not used in the Test ("remaining sample") and the test result(s) of the Test of the associated sample may help in further research, development and improvement of diagnostic methods and possible therapeutic solutions. Such measures may in the future also enable and support medical advice and guidance to the Patient, e.g. diagnosis and/or treatment of potential diseases.
- 患者的剩餘樣本以及相關樣本的該檢測結果和患者的人口統計數據 (包括其在年齡和性別) 將會以匿名及無限形式儲存在 GenieBiome 以供內部質量監控/作業及促進科學研究和技術開發 (統稱 "研究")。剩餘樣本及/或這些匿名數據將有機會為前述之目的向外國醫生、科學家及 (製藥) 公司 (統稱 "外部人士") 予以提供及/或使用。GenieBiome will retain the Patient's remaining sample, the test result(s) of the Test of the associated sample, and the Patient's demographic information (including his age and gender) anonymously and for an indefinite period of time for internal quality control/operation and advancement in scientific research and technology development (collectively, "Research"). Access to and/or use of the remaining sample and/or such anonymised data might be granted to external physicians, scientists and pharmaceutical companies (collectively, "External Parties") for the said purposes.
- GenieBiome 不會就上述之剩餘樣本、相關樣本的剩餘物檢測結果或患者的人口統計數據之提供及/或使用而為患者提供任何補償。GenieBiome will not provide compensation to the Patient for the access to and/or use of the remaining sample, the test result(s) of the associated sample, or the Patient's demographic information as stated above.
- 任何於 GenieBiome 的數據庫內的數據一旦被匿名化，該些數據將無法被要求刪除。這是由於匿名數據無法被識別及追蹤的。Any data in GenieBiome's databases – once anonymised – cannot be destroyed upon request as it is unidentifiable and untraceable.
  - 本人不同意以於上方第 I 至 IV 條所述的方式儲存和使用本人的剩餘樣本、相關樣本的該檢測結果和人口統計數據。I disagree with the proposed use of my remaining sample, test result(s) of the Test, and demographic information as stated in I to IV above.
- 參與研究: GenieBiome Limited 希望使用患者的聯絡資料 (包括姓名、電郵地址、聯絡電話號碼和地址 (如有提供))、透過電郵、手機短訊 (SMS)、即時通訊 (如 WhatsApp)、電話及/或郵寄, 不向患者發送或通知有關 GenieBiome Limited 及其子公司的產品及/或服務的推廣和直接促銷。GenieBiome Limited intends to use the Patient's contact details (including his name, email address, contact phone number and address (if provided)) for sending to or notifying the Patient of materials relating to the research of GenieBiome Limited and/or its subsidiaries, and for inviting the Patient for joining the research.
  - 本人不同意以於上方第 V 條所述的方式使用本人的個人資料作邀請參與研究用途。I disagree with the proposed use of my personal data for research purposes as stated in V above.
- 直接促銷: GenieBiome Limited 希望使用患者的聯絡資料 (包括姓名、電郵地址、聯絡電話號碼和地址 (如有提供))、透過電郵、手機短訊 (SMS)、即時通訊 (如 WhatsApp)、電話及/或郵寄, 不向患者發送或通知有關 GenieBiome Limited 及其子公司的產品及/或服務的推廣和直接促銷。GenieBiome Limited intends to use the Patient's contact details (including his name, email address, contact phone number and address (if provided)) for sending to or notifying the Patient of direct marketing materials relating to the products and/or services of GenieBiome Limited and/or its subsidiaries. GenieBiome Limited may not use a Patient's personal data in direct marketing or transfer or disclose a Patient's personal data to any third party (including its subsidiaries) for use by such third party in direct marketing unless it has received the Patient's consent.
  - 本人不同意以於上方第 VI 條所述的方式使用本人的個人資料作直接促銷推廣用途。I disagree with the proposed use of my personal data for direct marketing purposes as stated in VI above.

患者可以書面方式發送至上 "收獲個人資料聲明" 部份所列出的地址或電郵地址, 或透過 GenieBiome Limited 不附公佈的其他渠道, 表示拒絕 GenieBiome Limited 在直接促銷推廣上使用其資料, 並毋須承擔任何費用。The Patient may provide GenieBiome Limited with his objection to use his personal data in direct marketing in writing to the address or the email address stated in the section entitled "Personal Information Collection Statement" above or through other channels as GenieBiome Limited may announce from time to time at no cost to the Patient.

## 患者資料 Patient Information (必填, 資料與同樣的證明文件上的一致。Mandatory, please provide information that identical to the information on the identity document)

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地址 Address:

本人閱讀、明白、完全同意及接受本同意書內所載的所有條款及條件。  
I have read, understood, fully agree with, and accept all the terms and conditions set out in this consent form.

患者簽署 Patient Signature\*:  日期 Date\*:

樣本採樣日子 Sample Collection Date:  樣本採樣時間 Sample Collection time:

## 病歷 Medical History

1. 患者以往曾否接受過大腸鏡檢查?  
Have patient had a colonoscopy screening before?  
 有 Yes  沒有 No

(如有, 請回答以下兩個問題 if yes, please answer the following 2 questions)

2. 患者對上一次大腸鏡檢查是在多久前進行?  
When was patient's last colonoscopy screening?  
年前 Years ago

3. 患者在上一次大腸鏡檢查中, 有沒有發現癌肉並將其切除?  
Was any adenoma removed during the last colonoscopy screening?  
 有 Yes  沒有 No  不清楚 Not sure

(如有, 請回答以下兩個問題 if yes, please answer the following 2 questions)

4. 切除出來的癌肉在病理報告上是屬於什麼狀況?  
What was the pathological stage of the removed adenoma?  
 不清楚 Not sure

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Valid until 04-2024		<b>DHL</b> Charge to Receiver	
From (Shipper)		Payer Account Number: 639975078	
* Contact: _____ Phone: _____		Optional Service (extra charges may apply)	
		<input type="checkbox"/> Extended Liability	
* To (Receiver) GENIEBIOME (DIAGNOSTIC) CO LTD RM 902 9FL 208 JOHNSTON RD WAN CHAI		Weight	
		1 pcs kg	
Contact: Content / Remarks:		Phone:	
Track this Shipment: Website: dhl.com Hotline: 2710 8111			

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